

WIFI capability provides you an added option for data communication together with bi-directional LIS, USB port and LAN port, barcode reader, printer and keyboard.

BC-20s Auto Hematology Analyzer

Technical Specifications

Principles

Impedance method for WBC, RBC and PLT counting Cyanide free reagent for hemoglobin test

| Performance | | | |
|--------------------------|-----------------|------------------|--------|
| Parameter | Linearity Range | Precision (CV %) | Carryo |
| WBC(10 ⁹ /L) | 0-100 | ≤3.5% (4.0-6.9) | ≤0.5% |
| | | ≤2.0% (7.0-15.0) | |
| RBC(10 ¹² /L) | 0-8.00 | ≤1.5% (3.5-6.5) | ≤0.5% |
| HGB(g/L) | 0-280 | ≤1.5% (100-180) | ≤0.5% |
| MCV(fL) | | ≤1.0% (70-110) | |
| PLT(10 ⁹ /L) | 0-1000 | ≤5.0% (100-149) | ≤1.0% |
| | | ≤4.0% (150-500) | |

Parameters

19 parameters: WBC, Lymph#, Mid#, Gran#, Lymph%, Mid%, Gran%, RBC, HGB, HCT, MCV, MCH, MCHC, RDW-CV, RDW-SD, PLT, MPV, PDW, PCT 3 histograms for WBC, RBC and PLT

Reagent M-30D DILUENT M-30CFL LYSE PROBE CLEANSER

Sample Volume

20μL Prediluted mode Whole blood mode

Throughput 40 samples per hour

Display

8.4 inch TFT Touch Screen

Mindray Building, Keji 12th Road South,

Tel: +86 755 8188 8998 Fax: +86 755 26582680

E-mail: intl-market@mindray.com www.mindray.com

High-tech Industrial Park, Nanshan, Shenzhen 518057, P.R. China

Mindray is listed on the NYSE under the symbol "MR"

Data Storage Capacity

Up to 200,000 results including numeric and graphical information

Communication

LAN Port supports HL7 protocol Support bi-directional LIS

4 USB port (for external printer, software upgrade, barcode reader, WIFI adapter, keyboard and mouse), LAN port (1)

Thermal recorder, 50 mm wide paper, various printouts formats External printer optional

Humidity: 10%~90% Air pressure: 70kPa ~ 106kPa

Power Requirement 100V-240V ≤300VA 50Hz/60Hz

Dimension and Weight

Dimension: Depth(410 mm) x width(300 mm) x height(400 mm)

Weight: ≤20Kg

P/N: ENG-BC-20s-210285x6-20150331



BC-20s

Auto Hematology Analyzer

Minimum Size, Maximum Capability







What a 3-part should be

At Mindray we pride ourselves in our dedication and experience in developing better solutions for small labs. Our new line of 3-part hematology analyzers is the culmination of that effort. Compact yet powerful, full featured yet affordable, the BC-20s is what a 3-part analyzer should be.



Exclusive Feature

Detailed flag information never before seen on a 3-part analyzer. Provides information useful for diagnosis including WBC flag, RBC flag and PLT flag.

BC-20s

Auto Hematology Analyzer

Sample 1: BC-20s shows flag "Lymph increased" which means a high number of Lymphocytes and/or immature cells. Meanwhile R1 flag is also displayed. Two kinds of flag messages are both supported to ensure clinicians have better understanding of sample results.

Sample 2: Flag "Anemia" and "HGB Abn./Interfere?" are displayed. They mean the sample has shows either signs of Anemia, Abnormal Hemoglobin or sample is interfered such as by high value of Leukocytes or Agglutinated Erythrocytes.

Sample 3: Flag "Thrombopenia" indicator is shown together with Platelets low flag.

Different flag information provided according to parameter results together with histograms.



Better Usability

Minimum size with the footprint similar to that of a 17 inch laptop, with space saving design that allows internal storage of lyse giving small labs more space. 8.4 inch TFT touch screen together with our powerful software enhances user operations and experience.

Higher Efficiency

New technology that eliminates the need for cleanser and rinse, reducing the number of reagents needed while at the same time lowering overall reagent con-

Flexible packaging of reagents, with normal and small sizes to better cater to the needs of different daily sample volumes.







Enhanced Performance

Higher throughput at 40 tests per hour.

